**HDLc/LDLc CALIBRATOR**

Human Serum  
Only for in vitro use in clinical lab  
Store at 2-8°C

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4x1 mL

**PRODUCT CHARACTERISTICS**

HDLc/LDLc calibrator is a human lyophilised serum.  
Is intended for determination of HDLc or/and LDLc concentrations with manual and automated analytical procedures.

**REAGENTS**

Human serum. Biological additives.  
The concentration of the components are lot-specific.  
See chart below for the values of ingredients.

Lot: LD0951108

Exp.: 2024-06

**PRECAUTIONS**

Components from human origin have been tested and found negative for the presence of HBsAg, HCV and antibody to HIV (1/2). However handle cautiously as potentially infectious.

**CALIBRATION**

Information available on request.

**PREPARATION**

-Open one vial carefully, without lost of the lyophilised  
-Reconstitute (->) with 1,0 mL of distilled water.  
-Mix thoroughly, avoiding foam forming.  
-Bring to room temperature for about 30 min. before use.  
-Improper handling and/or storage can affect results.  
-Inaccurate reconstitution and errors in assay technique can cause erroneous results.

**STORAGE AND STABILITY**

The Control serum is stable until the expiration date on the label when stored tightly closed at 2-8°C and contaminations are prevented during their use. Do not use reagents over the expiration date or if there is visible evidence of microbial growth.

Store tightly capped when not use.

- After reconstitution is stable for:

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